


Mounting Description

安装说明

Abduction block for Sliding seat 分腿器-（供滑座板型用）

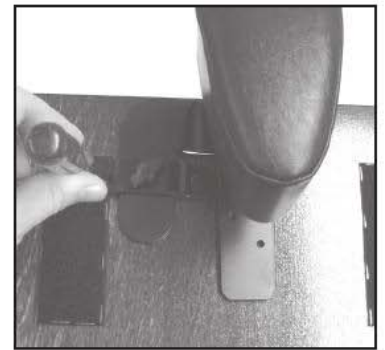

Tools: awl, 6mm drill & star screwdriver

工具: 锥子, 6 毫米钻头以及星形螺丝起子

1. Place the abduction block in center and on top of the seat plate, and find the required position for the locking adapter.



把分腿器放在座板的中央上部，将连接件放置在要求的位置

Make marks, using an awl, for the screw holes of the adapter, which is to be mounted on the **UNDERSIDE** of the seat plate.

用锥子标出分腿器连接件的螺丝孔位置。分腿器连接件应安装于座板下方。

2. Drill two holes through the seat plate. Fix the adapter on the **UNDERSIDE** mounting the screws through the top of the seat plate using a star screwdriver.

在座板上钻出两个小孔。用星形螺丝起子将螺丝从座板上穿入后，将连接件固定在座板下面。

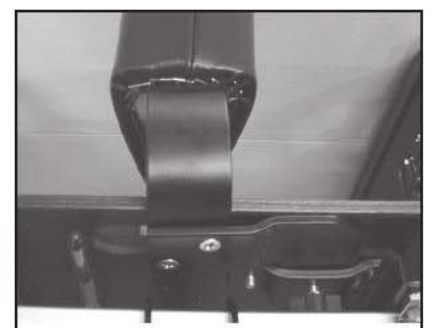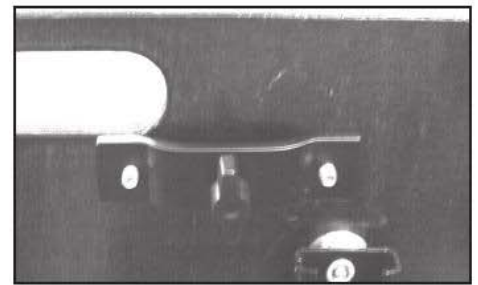

3. The depth of the abduction block can be adjusted by moving the star screws on the underside.

调节座椅板下面的星轮螺丝可调节分腿器深度

4. Mount the abduction block by sliding the right bar into the adapter. Fix it by tightening the wing screw.

将分腿器右侧杆推入连接件中,旋紧翼形旋钮将其固定

Item no.	Item
22103	Abduction block for sliding seat Small 60 小号分腿器-滑座板
22104	Abduction block for sliding seat Medium 70 中号分腿器-滑座板
22105	Abduction block for sliding seat Large 80 大号分腿器-滑座板