


## Mounting Description

### 安装说明

### Abduction block 分腿器

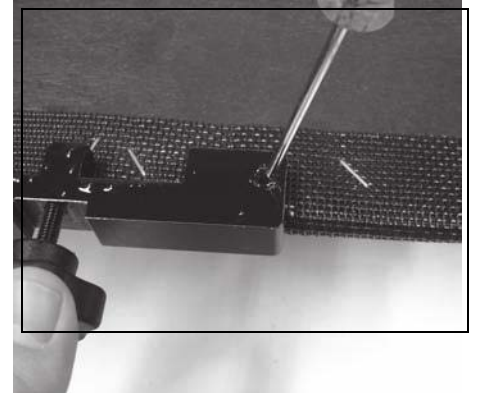

Tools: awl, 6mm drill & 4mm Allen key

工具: 锥子, 6 毫米钻机以及 4MM 内六角扳手

1 Place the locking bracket in center of the seat plate.

将分腿器连接件放置在座板中央

2 Use an awl to mark the position of the locking bracket screw holes.

The locking bracket is to be mounted on the **UNDERSIDE** of the seat plate.

用锥子标出分腿器连接件的螺丝孔位置。分腿器连接件应安装于座椅板下面

3. Drill two holes in the seat plate and mount the locking bracket on the underside of the seat plate. Mount it by fixing the screws from the top of the seat plate and down do using a 4mm Allen key.

在座板上钻出两个小孔, 将螺丝从座板上穿入后, 用六角扳手将连接件固定在座板下面

4. Mount the abduction block by placing the adjustment tube into the locking wheel of the bracket. Fix the abduction block by turning the locking wheel.

将调节杆置入) 连接件锁紧轮中, 扭紧锁紧轮将分腿器固定。



5. The abduction block can be adjusted 8cm in height (5B) to be compatible with cushions used in the market.

可根据市场使用的坐垫高度调节分腿器高度, 调节范围为 8CM,

The abduction block can be adjusted 10cm in depth (5A) for individual adjustment.

分腿器深度调节范围为 10CM.

# ALU\_REHAB

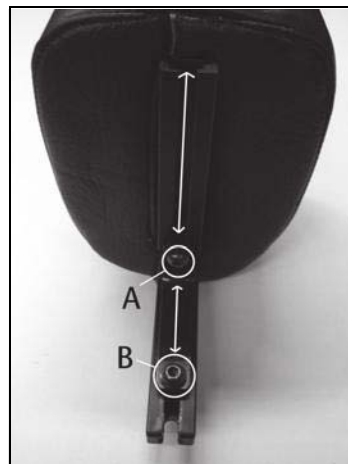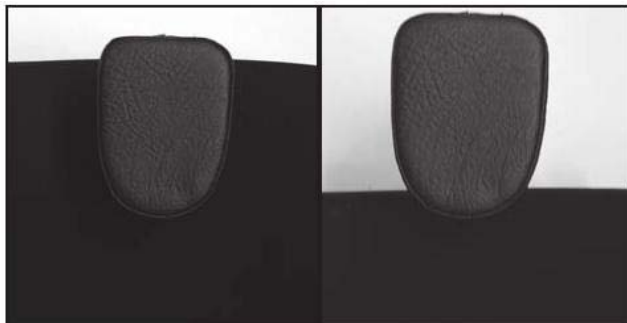

Item number 编码	Item 名称
27490	Abduction block Small 小号 80mm
27491	Abduction block Medium 中号 110mm
27492	Abduction block Large 大号 140mm