


Mounting Description Brake 刹车安装说明

Tools: 5mm Allen key

工具: 5MM 内六角

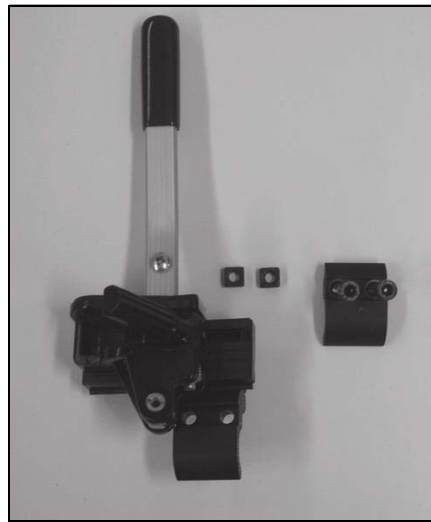

1.

1. Place the brake clamp on the top frame tube.

Insert the square nuts into the trace of the brake clamp.

将刹车卡件安装在边框上部支架管上. 将方形螺母放入刹车卡件内

2. Mount the other clamp half by fixing the screws to the square nuts.

用螺丝将另一半卡件与方形螺母固定

3. There should be a distance of app. 15mm between the brake and the main wheel in open position.


刹车放开时与大轮距离约为 15MM

4. Push the handle forward to activate the brake.

刹车向前推为使用状态

5. Pull the handle rearward to open the brake.

刹车手柄向后拉为放开状态

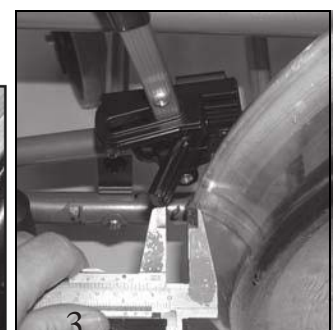

4.

5.

3.