


Mounting Description

Side Support Stable

稳定型侧部支撑安装说明

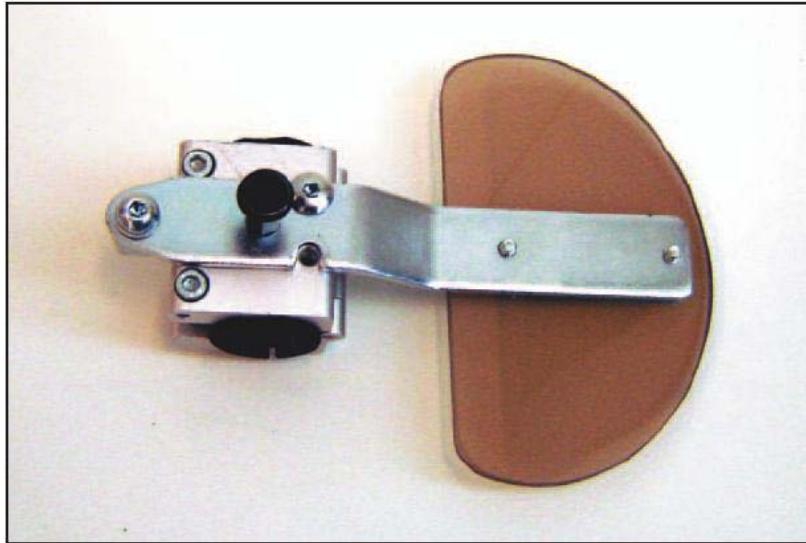

Item	Item number
Side support Stable Left	26830
Side Support Stable Right	26843

Tools: 6 mm Unbrako og en skarp kniv


6mm内六角扳手

Take out the two screws holding the side support clamp together, & open the clamp.

将侧部支撑卡件螺丝松开，打开抱夹。

If the side support is going to be mounted on a wheelchair with square backrest tubes use the enclosed plastic bushing.

若侧部支撑需安装在长方形靠背支架管上,可用所配塑料环套.

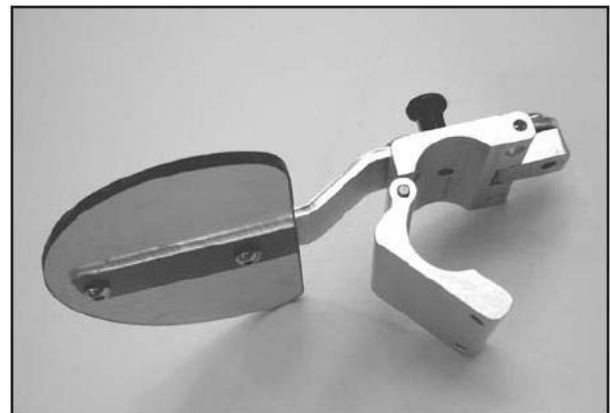


After fitting the backrest cushion find the right height for the side support. The support should be parallel with the integrated side support of the backrest cushion. Cut out a piece of the velcro backrest to give room for the clamp to fit around the backrest tube.

放入靠垫,找到安装侧部支撑的合适位置. 侧部支撑高度应与靠垫侧部支撑平行. 在尼龙靠背上剪开一小口,以便安装侧部支撑支架

When mounting on square tube, place the enclosed plastic bushing around the backrest tube first.

在安装于方形支架管上时,请先在外部套上塑料环套

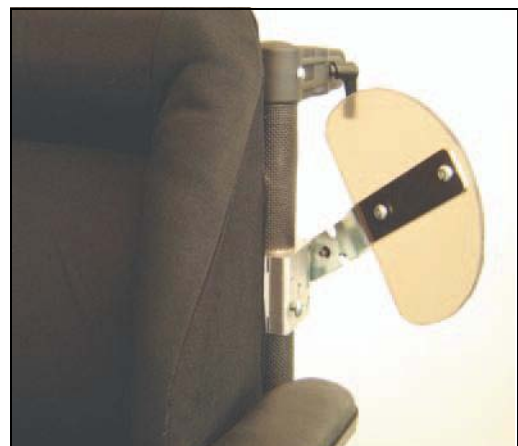

The clamp is mounted & the screws tightened. Before the screws are tightened, the clamp is rotated around the backrest tube to increase or decrease stability.

安装抱夹并锁紧螺丝。在没锁紧螺丝前，抱夹会沿靠背管上下左右窜动不稳。
调节支架卡件螺丝松紧可增加或减少其稳定程度。

To release the side support, pull the release button out.

拉出锁闩拉钮，可松开侧部支撑。