

## Monteringsbeskrivelse ryggforlenger.

Løsne skruene som vist på bilde 1 og ta ut firkantmutter inne i ryggrøret.

Bilde 2 viser ryggforlenger med firkantmutter.

Sett deretter ryggforlengerne ned i ryggrørene og søk med skruen slik at skruen festes i mutter som sitter i ryggforlengerne og stram skruen som vist på bilde 3.

Plasser deretter tekstilet med velcroen på fremsiden slik at den er parallell med velcro på standard ryggen.

Løsne tekstilene fra hverandre og strekk disse slik at de har samme form/bue som velcro på rygg har.

Etter at tekstilet med velcro er innstilt på ønsket bredde så kan den klippes på utsiden av sømmene på hver del.



## Mounting description backrest extender.

### 靠背延长杆安装说明

Unfix the screws as shown on picture 1 and remove the square nut inside the backrest tubes.

松开图片 1 中所示螺丝,取出靠背管内的方形螺母

Picture 2 shows the backrest extender with the square nut.

图片 2 为靠背延长杆及方形螺母

Place the bckrest extender in the backrest tubes and fin the screw in the square nut of the backrest extender as shown on picture 3.

如图 3 所示,将靠背延长杆插入靠背支架管中,用螺丝固定住靠背延长杆中的方形螺母

Place the fabric onto the backrest extenders so that the velcro is equal with the velcro on the standard backrest.

将靠背绷布套入延长杆上,使其与标配靠背保持水平.