

Mounting description for
Recline release handle Netti Svipp for armrest.

Svipp 扶手用倾斜调节手柄安装说明


To fix the release handle, you must unfix the foremost screw under the armrest pads.

安装手柄时,先松开该扶手垫下条最前端的螺丝

Put a M5x35 screw through the fixing hole on the release handle, and mount it under the Armrest pad with the existing screw

将 M5x35 螺丝穿入该手柄中的固定孔后固定在扶手垫下方

Fixing hole 固定孔位(右侧)

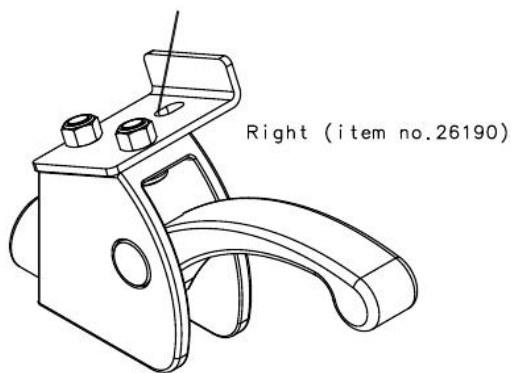

Fixing hole

固定孔位(左侧)

