

Monteringsbeskrivelse Hoftebelte Netti Svipp

Mounting description Hip belt Netti Svipp

Svipp 臀部保护带安装说明




1



Norsk

Trekk beltet gjennom hoftebeltefestet (Bilde 1) og tre deretter beltet tilbake gjennom belteklemmen (Bilde 2 og 3).

Plasser foring på innsiden av bakhjulsbraketten (Bilde 4) og tre skruen gjennom hoftebeltefeste og inn i bakhjulsbraketten (Bilde 5). Plasser mutter på innsiden og stram skruen. Pass på at hoftebeltefestet ligger 45° oppover (Bilde 6). Dette gjøres på begge sider.

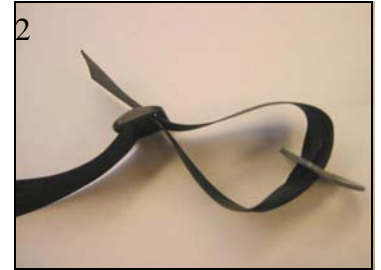


3



Juster deretter lengden på beltet ved hjelp av justering i belteklemme (Bilde 3) på hver side.

6 English

Tread the belt through the hip belt bracket (Pic. 1) and then tread the belt back through the beltclamp (Pic. 2 and 3)
将臀部带扣穿入带子后(图 1), 再反穿入带夹里(图 2,图 3)

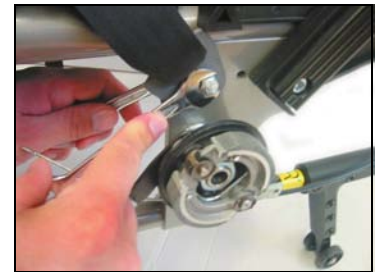

7

Place the sleeve on the inside of the main wheel bracket (Pic. 4) and thread the screw through the hip belt bracket and through the main wheel bracket (Pic 5). Put the nut on the inside and fasten good and make sure that the hip belt bracket is in a 45° angle (Pic. 6). Repeat the same operation on the other side.

将螺丝套装入主轮轴套孔里(图 4)并把螺丝穿入臀部带扣和螺丝套里(图 5), 在内边旋入螺母并锁好, 确认该臀部带扣成 45 度角(图 6)。

另一侧方法同上。

Adjust the length on the hip belt using the belt clamp (Pic 3) on each side.

在两边调好臀部处带长并用带扣锁好(图 3)。



